

DATA SHEET

ANIMALINTEX

DOSAGE AND METHOD OF USE

- Thoroughly clean the area to be treated
- place the Animalintex flap in a shallow bowl, with the plasticised surface facing outwards
- Pour water into the container around the edges of Animalintex
- After it has soaked in water, remove the pack from the basin and remove the excess water with gentle pressure
- Apply the pack to the affected area with the plasticised surface outwards (not in contact with the skin), shaping it around the edges of the application area
- Apply an external restraining bandage, ensuring that the wrap is completely covered



PACKAGING

One envelope contains 1 flap

One box contains 10 flaps

DESCRIPTION

Multi-layered, highly absorbent pack, easy to use and suitable for a wide range of common horse problems.

- *Hot pack*: abrasions, bruises, contaminated and infected wounds, abscesses
- *Cold pack*: tendonitis, desmitis, schinelle, trauma

PRODUCT CODE

EAN13: 5010058057043

